

REALISTIC SILICONE SKIN COVERS

CARE: ESSENTIAL MAINTENANCE TIPS

The SkinTones System: Protecting Your Prosthesis with a Natural Look. Our skins offer a durable, waterproof barrier for your device. For extended life, read the care instructions carefully. Questions? Visit our FAQ at getskintones.com or contact us.

Cleaning & Care for Silicone Skin Prosthetics:

- **Routine Cleaning:** Use lukewarm water with Dawn or any unscented, color-free dish soap to remove daily dirt and oils.
- **Tough Stains:** For stubborn stains, apply a small amount of alcohol and gently scrub with a kitchen pad. Be gentle to avoid surface damage.
- **Nail Polish on Toenails:** You may paint the toenails with nail polish – YOU MUST USE ACETONE-FREE NAIL POLISH REMOVER
- **Prevent Staining from Clothing & Shoes:** New clothes and dyed leather shoes may stain the skin. Wash or dry-clean new garments before wearing, and be cautious with dyed leather footwear to prevent irreversible staining.

Important Care Guidelines for Your Protective Silicone Skin Cover:

- **Footwear Caution:** Always wear solid-soled shoes. Avoid high-impact sports or activities while using the silicone skin cover to prevent damage.
- **Product Restrictions:** Do not apply self-tanners, lotions, creams, hand sanitizers, or any cosmetics to the silicone skin. Additionally, avoid using anti-bacterial soap as it can harm the material.
- **Oil Exposure:** Keep all oil-based products away from the silicone skin. In case of accidental contact, immediately clean the area with a mixture of Dawn dish detergent and water, or use rubbing alcohol to wipe it down.

Adhering to these guidelines will help maintain the integrity and appearance of your protective silicone skin cover.

